



HOOF-it® Pad Directions:



1. Thoroughly clean the sole of hoof, remove all debris and insure the hoof is dry.
2. Mix HOOF-it® II powder with liquid in the measuring / mixing cup to a pourable consistency.
3. Pour half the HOOF-it® II mixture onto the hoof sole.
4. Using the applicator stick and wet hands press the HOOF-it® II resin onto sole, forming a seal where the shoe and sole meet.
5. Now pour the remaining HOOF-it® II mixture on top to fill the rest of the sole area.
6. Set down leg on a clean surface and allow the HOOF-it® II Acrylic pad to cure for approximately 2 minutes.

Note: At outside temperatures above 75 F mix more liquid for a thinner mixture.
Ready to ride, stable or pasture.

Note: The HOOF-it II pad can be removed at any time with a hoof pick or by softening the HOOF-it II acrylic with acetone if necessary.