



HOOF-it® Composite Horseshoes Directions:



1. Trim the hoof for balance
 2. Rasp to level using hoof rasp.
 3. Use a gauge to insure the correct hoof angle.
 4. Start fitting the HOOF-it® Composite. When choosing the correct size shoe for your horse, you might want the shoe to be somewhat larger than the hoof; the excess will be rasped off later (better a little larger than to small, and remember you want to fit the shoe to the hoof, not the other way around...) **NOTE:** to remove the toe clip, just rasp it off.
 5. Use 2 or 4 nails (use standard nails) to nail the shoe in place (Less nails equal a healthier hoof!) **Note:** You can nail through the material or pre-drill holes – your choice.
 6. Now rasp the excess shoe width off, to custom fit the shoe to the hoof.
 7. Additional nails can be used if needed.
 8. Clinch the nails to apply the finishing touches. Cover the nail holes and the hoof with a protective HOOF-it® II coat.
- Ready to **ride, stable or pasture.**