

Hoof Glue Pad



- 1) Trim and balance the foot.
- 2) Clean the sole surface with denatured alcohol, paying close attention to the frog area, and allow to dry.
- 3) Shape the shoe, and concave (safe) the inside edge of the shoe. (Note: Because of Quick-Pads superior bonding strength, it is not necessary to use any type of mesh.)
- 4) Nail on shoe and clinch.
- 5) With the foot elevated, apply Quick-Pad to the sole of the foot until you have the desired thickness.
- 6) Quick-Pad will cure in 30-40 seconds. If Quick-Pad is still tacky, cover with plastic wrap or baby powder, and put the foot down. (This prevents Quick-Pad from bonding with the floor.)

*Quick-Pad may also be used under a pad.

*Quick-Pad may also be used in smaller applications to protect the sole from trauma, and prevent abscessing. By applying $\frac{1}{4}$ " of Quick-Pad to the sole of the foot you can prevent harmful bacteria from invading the frog, and prevent thrush.
